

Axis Swivel Wheel Adapter

In order to install the swivel wheels first you must remove the standard front wheel by unfolding the quick release lever and sliding the wheel out of the fork tips.


Next loosen the black knob on the top of the Axis swivel wheel adaptor and unfold the hinged lid.


Now slide both forks into the Axis Swivel Wheel Adapter until the Fork Tips slide over the round cross bar.

Finally fold the hinged flap closed and turn the black knob until snug. Give the Axis Swivel Wheel kit a tug and make sure it is on tightly.



Quick Release Lever

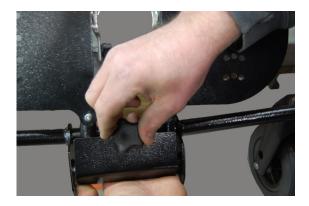

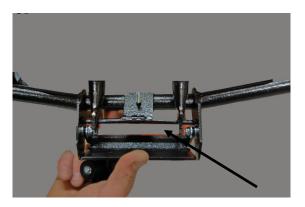

PLEASE NOTE: Make sure the Axis Wheel Adapter is properly installed before each use. Cross bar

www.adaptivestar.com