



www.adaptivestar.com

1-800-278-9626

PO Box 369 Selah Washington 98942

Installation of Anti-tippers on an Adaptive Star Axiom Push Chair



Start by removing all of the clamp halves (as shown on the right) with the provided Allen wrench

Next insert one of the anti tippers between the 2 cross bars on the axle assembly or the Axiom Push Chair. The axle assembly is the portion of the chair that the rear wheels attach to. Now reinstall the clamp halves around the frame tubes and tighten the bolts with the Allen wrench provided. Repeat to install the second anti-tipper.



The final installation should look like the photo, left.

If you have any questions, please call us! we're happy to help

1-800-278-9626